## CIRUS Nano

### Definition

Light curing nano-hybrid composite, for anterior and posterior restorations. Highly filled. Restorations of class I, II, III, IV and V.

### Composition

Organic matrix: Bis-GMA - TEGDMA

Fillers: Baryum glass particles, silanised (80%), nano silicate fillers.

### Safety notes

#### Warnings

Cirus Nano contains polymerizable methacrylates which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.

Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.

Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

#### Precautions

Wear suitable protective eyewear, mask, clothing and gloves.

Protective eyewear is recommended for patients.

### Contraindications

Cirus Nano is contraindicated in patients with a history of allergic reaction to methacrylate resins or any other of the components. Do not use Cirus Nano in cases of pulpar exposition, pulpitis.

### Instruction for use

- Select the appropriate VITA shade with day-light.
- Prepare the tooth cavity in a conventional manner. The margins are prepared by slightly beveling the occlusal surfaces and producing slight flares on interproximal cavities.
- For classes II, III and IV, a matrix band is placed and secured by wedges; it should be loosened to allow full contours when packing the restorative material into the cavity.
- When the cavity is closed to the pulp, apply a eugenol-free cavity liner (ex: calcium hydroxide, antibacterial liner).
- Etch the preparation, following manufacturer's instructions and apply the light curing bonding agent of your choice. Or use a self-etching bonding agent.
- Apply Cirus Nano and light-cure in layer not exceeding 2,5mm thickness, to minimize the retraction of the material. Exposure time is 40s for layers of 2,5 mm, in light shades (A1, A2,A3, C2, etc) and 60 s for darker shades (A3,5, B3, etc).
- After matrix band and wedge are removed, light cure the material from buccal and lingual sides, as well.
- Finish and polish the restoration.

### Storage & shelf-life

Do not expose the product to sun-light or warm sources. Never store CIRUS NANO close to products containing eugenol.

Store at a temperature not exceeding 23°C. When stored under such conditions, Cirus Nano has a shelf-life of 36 months.

### Presentations & references

**CIRN-4,5\*** - 4.5g syringe. Available in the following VITA shades: A1/B1, A2, A3, A3,5, A4, B2, B3, C2.

**CIRN-5\*** - Box of 20 x 025 g compules, available in the following VITA shades: A2, A3, A3,5, B2.

\*mentioned the shade.

### Important notice

- Do not throw away the box and the instruction for use until the product is completely finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

### Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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