**A** **DIAGNOSTIK**  
DIAGNOSTICS  
DIAGNOSTIC **A.01 – A.12**

**B** **KONSERVIERENDE BEHANDLUNG**  
RESTORATIVE  
SOINS CONSERVATEURS **B.01 – B.16**

**C** **NICHTCHIRURGISCHE PA-THERAPIE**  
PERIODONTAL  
PARODONTOLOGIE **C.01 – C.22**

**D** **ENDODONTIE**  
ENDODONTIC  
ENDODONTIE **D.01 – D.10**

**E** **EXTRAKTION**  
EXTRACTION  
EXTRACTION **E.01 – E.74**

**F** **PARODONTAL CHIRURGIE**  
PERIODONTAL SURGERY  
CHIRURGIE PARODONTAIRE **F.01 – F.18**

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**I** **PRAXIS ORGANISATION | GPM**  
PRACTICE ORGANIZATION | GPM  
ORGANISATION DU CABINET | GPM **I.01 – I.24**

**X** **INHALTSVERZEICHNIS**  
INDEX  
INDEX **01 – 22**

**F** PARODONTAL CHIRURGIE  
PERIODONTAL SURGERY  
CHIRURGIE PARODONTAIRE

 **deve**  
**med.**  
PARTNER  
FOR DENTAL  
INSTRUMENTS

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For the initial incisions in gingivectomy and gingivoplastic treatments and especially suitable in retro-molar region.

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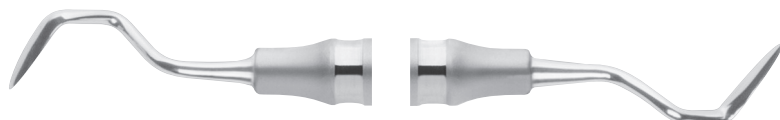
Also useful for excising interproximal tissue. Working area: Molars

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
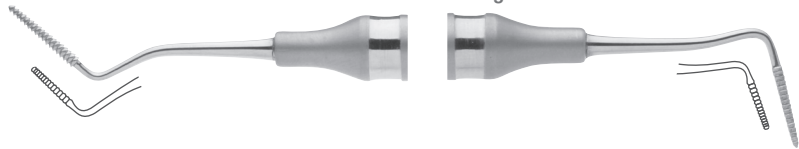
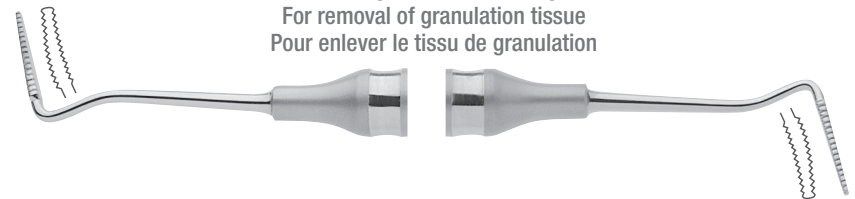
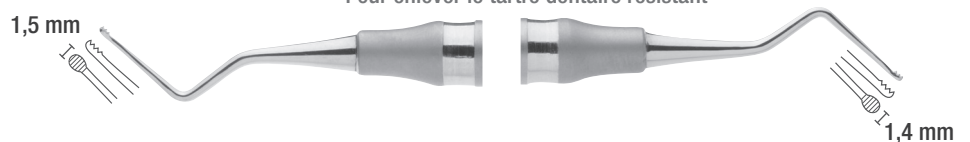
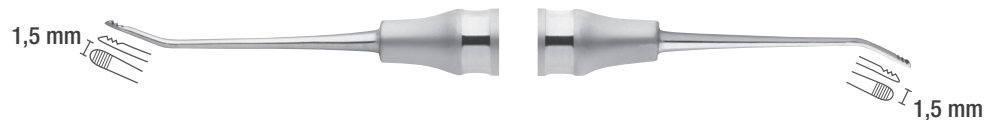
Curette for removal of hard subgingival deposits and granulation tissue

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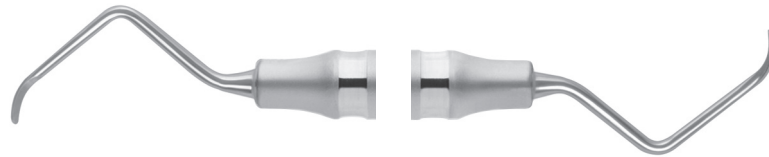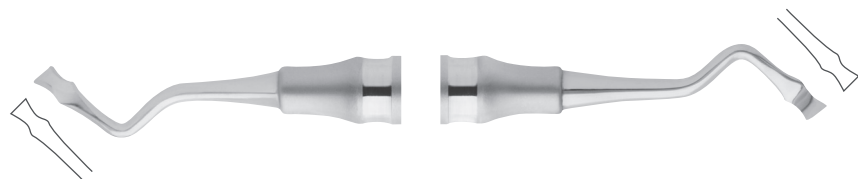
The chisel is recommended for atraumatic removal of bone material attached to the tooth and is particularly suited for the use at the rear distal molars. (pulling use)

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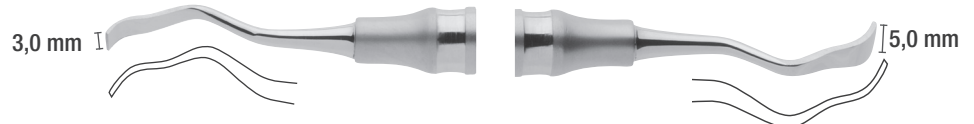
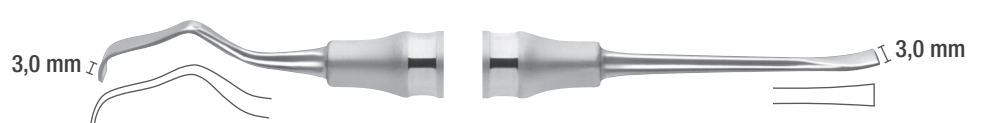
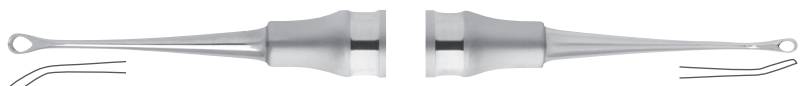
Extended field of view through the opening in the working part. For open and closed curettage, pulling use.

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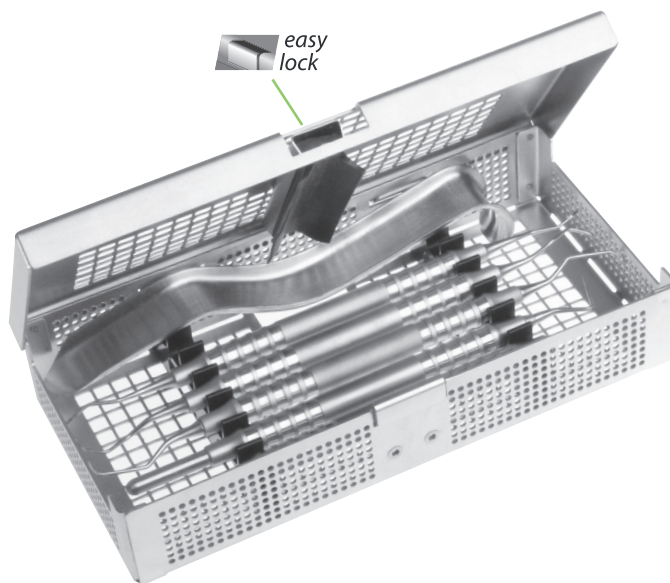
### Implant Maintenance | Implant-Prophylaxe

Scientific studies have shown that cleaning titanium abutments with titanium instruments causes no change whatsoever to the surfaces. Therefore our implant prophylaxis instruments are manufactured with pure, soft titanium. The workpiece ends correspond with our conventional curettes.

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For the cleaning of implants, as well as the removal of calculus and plaque, without damaging the titanium abutments.

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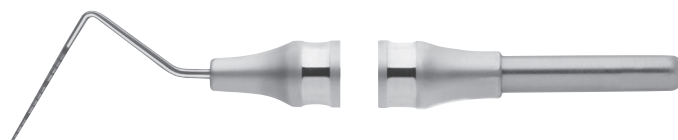
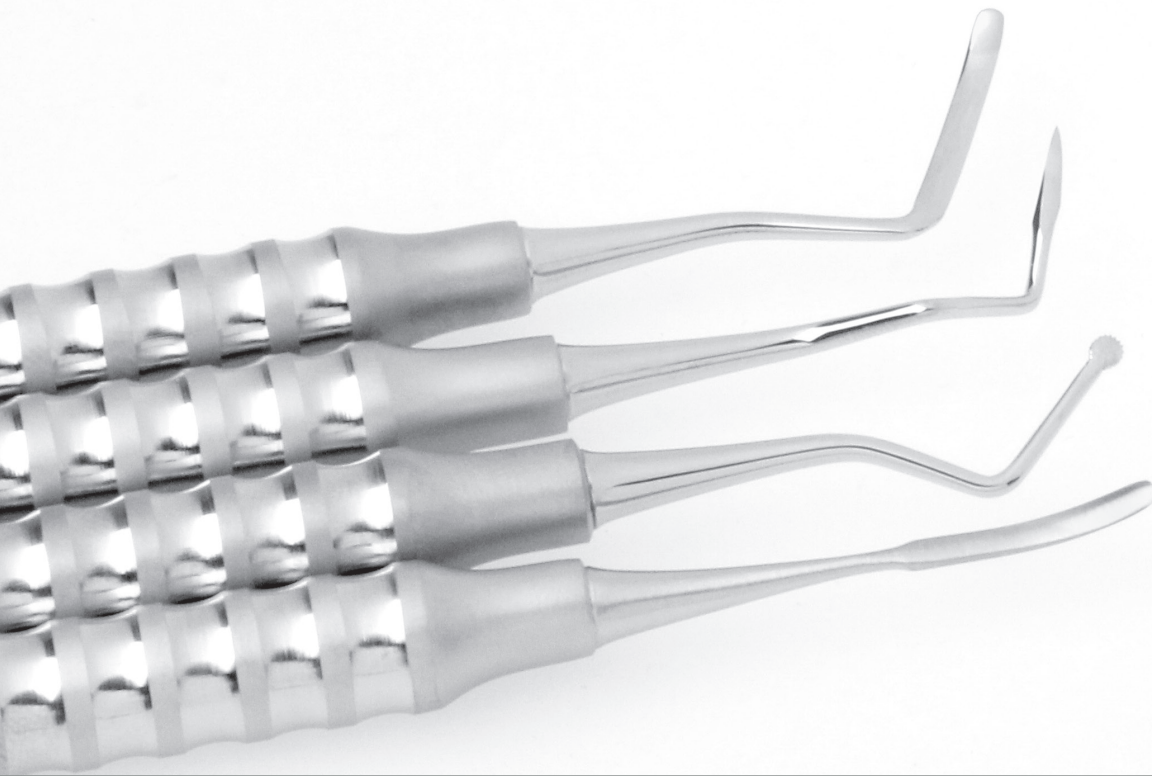
Lateral tunnel procedures are an approach which improves the aesthetics and their predictability, as they avoid loosening critical papillae and maintain high vascularization of the connective tissue graft.

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Starting at the Gingival mucosa, the attached mucous membrane can be surgically detached supra-periosteal with the slightly curved end of workpiece. The rounded tip prevents perforation of the mucosal flap. In the mobile mucosa area, the minimal invasive surgical removal can be continued with the straight, pointed end of workpiece.

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This instrument is an alternative to the PPS 1000 F and is used for the same application. The lineal, rounded work piece favours a more conservative working method.

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
The workpiece ends are arrow-head shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunnelled by using the very flat workpieces.

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In comparison to PPS 1200 F this instrument differs considerably in the curvature of its workpieces. With a bridge flap, this instrument can be optimally used to prepare tunnels from apical to coronal through an entry in the periodontal membrane.

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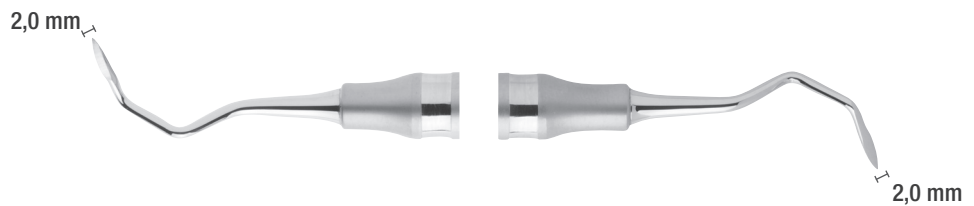
The mucosal flap preparer is particularly suitable for the posterior tooth area. Access and atraumatic mucosal flap preparation is possible due to the slightly curved workpieces.

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The mucous membrane transplant can be inserted into the prepared tunnel using this instrument.

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The lance-shaped end of the workpiece enables the detachment of the papilla as well as of the marginal gingiva crest. With the slightly curved working part the gingiva is suprapariostally removed/loosened - the rounding of the working part prevents the mucosal flap from being pierced.

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The workpiece ends are arrow-head shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunnelled by using the very flat workpieces. To avoid injury to the gingiva, the side of the working part which faces the gum is blunt.

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The gingivectomy knife can be used as a micro-blade for the tunnel technique due to its flexible, sharp working end.

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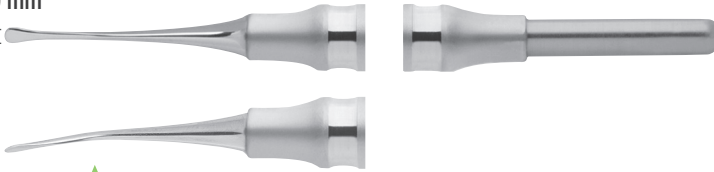
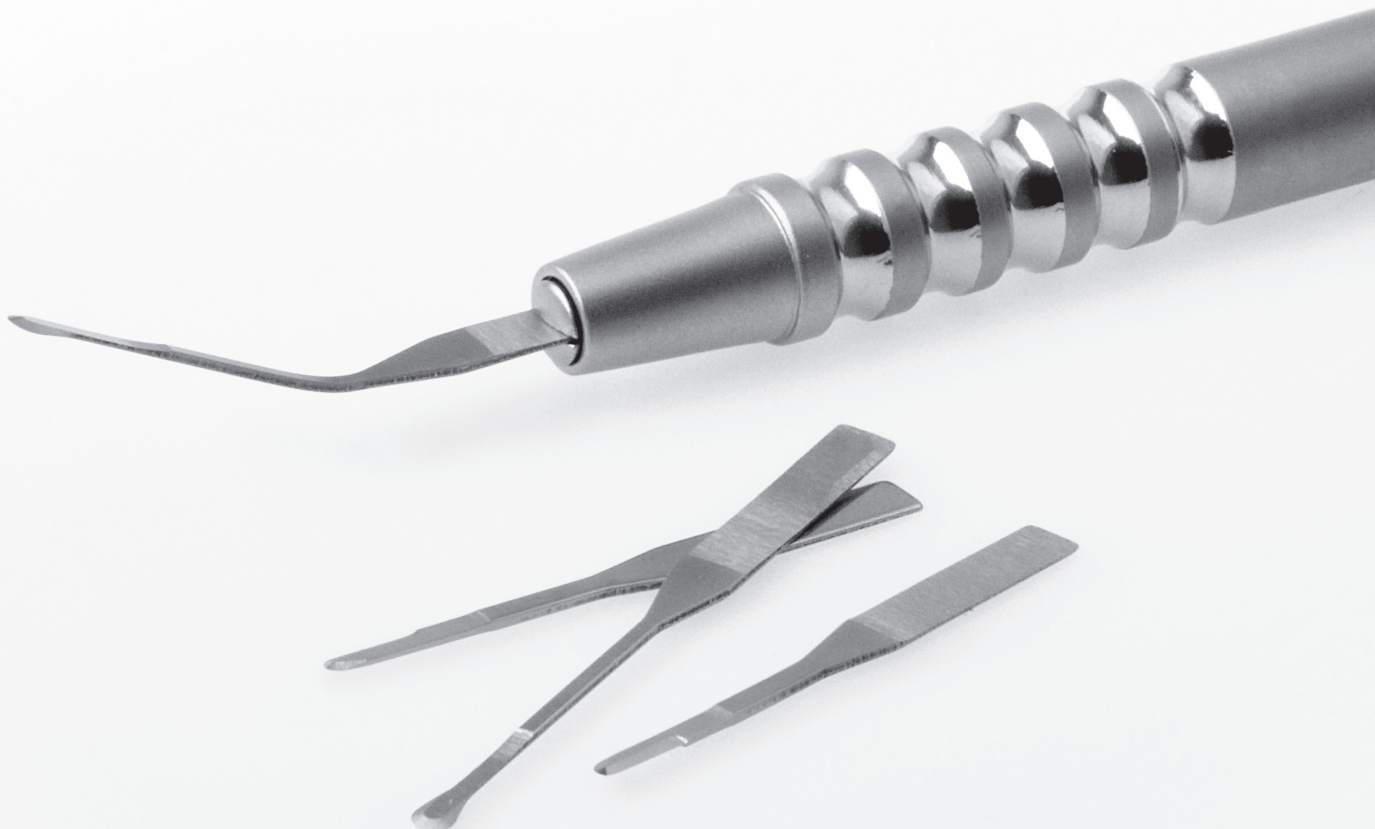
The micro blades can be brought into shape with a needle holder easily. They enable an easy work in hard to reach areas like mesial or distal region and interdental spaces and they are perfectly for minimal invasive treatments.

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All scalpel blades are made of stainless steel, individually sterile packed and are available in packages of 10 pcs.

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The blades has to be bent carefully with a needle holder

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**ATTENTION:** For regulatory reasons, this products cannot be shipped to every country!

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The filigree working parts allow even greater minimally invasive atraumatic working.

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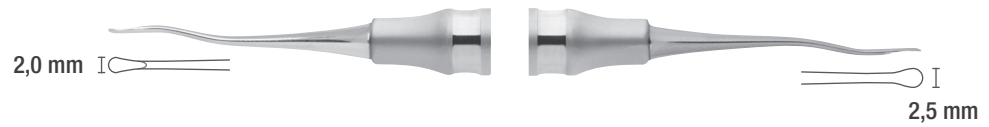
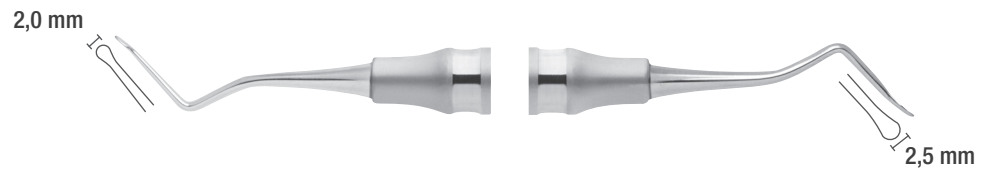
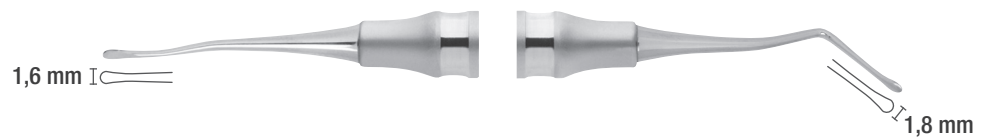
The periosteum can be loosened and expanded by short, pulling movements exerted with slight pressure. This technique ensures an optimal blood supply, reduces swelling as well as pain and minimizes injuries to the vessels.

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