



## STARJET \*

### ■ stop limitations

A typical scenario - you complete the supragingival air-polishing with your turbine-adapted handheld unit and your next step is subgingival cleaning but your unit is not intended for this use.

### ▶ start flexibility

With the starjet, these limitations no longer exist. With the flexibility of choosing the appropriate powder, you can easily switch from supra- to subgingival by simply turning the selection ring between prophy and perio. Not flexible enough? You can also select your preferred spray head, with three options.



120°

PERIO

90° (optional)



### ▶ 3 different spray heads

Always best access - 90° and 120° angled spray heads for supra- and subgingival use in periodontal pockets up to 5 mm depth.

The spray head PERIO equipped with the disposable sterile Subgingival Perio Tip provides optimal access in pockets deeper than 5 mm.

Just one click connects the spray head safely to the starjet, thanks to a dedicated security connection system.

\* patent pending



► **Subgingival Perio Tip**

Flexible and soft - the subgingival, disposable Perio Tip will come sterile and attaches easily to the spray head PERIO by hand.



► **exclusive twist ring for powder selection**

Reach the maximum flexibility as it can be used supra- and subgingival.



► **transparent powder chamber**

For the best powder level visibility.

► **easy filling**

Refills are clean and simple due to the lateral opening of the powder chamber. No balancing act or messy spills.



► **7 different turbine adapters**

- Bien Air Unifix and Unifix L
- KaVo multiflex and multiflex lux
- Sirona R and B
- W&H Roto Quick and Roto Quick lux
- NSK
- Midwest
- Borden





► **PROPHYLAXIS  
POWDER SMOOTH**

gently removes, thanks to its spherical shape, supragingival biofilm while respecting the surrounding tissues (calcium carbonate-based - average particle size between 45 µm and 75 µm)

Indications:

- Teeth cleaning to a shiny finish, while avoiding enamel damage
- Allows better access to the areas difficult to reach
- Aids in reducing the PH acidity in the mouth
- Reduction in aerosol effect and risk of device clogging
- Sodium and silicon (SI) free
- The absence of sodium makes it possible to treat patients maintaining a salt-free diet



► **PROPHYLAXIS  
POWDER SOFT**

gently and effectively removes supragingival biofilm and stains on the enamel surface (sodium bicarbonate-based - average particle size 40 µm)

Indications:

- Removal of stains and extrinsic discolorations due to smoke, wine, tea and coloring agents
- Cleaning of fissures and cavities
- Cleaning of orthodontic brackets



► **PROPHYLAXIS POWDER  
SENSITIVE – GLYCINE  
POWDER \*\***

gently removes supra and subgingival biofilm (glycine-based - average particle size 25 µm)

Indications:

- Mild pigmentation and stain removal
- Periodontal pocket detoxification
- Periodontal maintenance
- Implant maintenance and treatment of peri-implantitis
- Orthodontic patient maintenance
- Gentle cleaning of restoration materials and comfortable treatment on exposed root surfaces
- Reduction of bacteria colony-forming units (CFU)
- Can be successfully used for the treatment of each type of patient



\*\* available only in Europe